

## **Summer Lockbox Instructions**

During the warm months (April - October), follow these steps for loading the lockbox to prevent overheating:

Place a frozen cool pack in the bottom of the lockbox
Layer 4 paper towels over the cool-pack for insulation
Place refrigerated specimens on top of the paper towels
Layer 4 paper towels over the refrigerated specimens for insulation
Place ambient temperature specimens on top of the paper towels

Ambient temperature specimens
Paper towels
Refrigerated temperature specimens
Paper towels
Frozen cool pack on bottom